EXHIBIT P THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 5000*

STANDARD CALIBRATION CHECK PROCEDURE

		Date 4-13-03 Time 08 84 Detail # (0(2-00 4416 Location						
Intoxilyzer Serial # (0(a 00 44(b) Location								
QA Spe	cialis							
		(Print Name)						
(🗸)	1.	Pour a standard alcohol solution of known value into a clean dry simulator jar and assemble the simulator. Insure that a tight seal has been made. Standard value: 0. 100AC						
(\mathcal{N})	2.	Turn on the simulator and allow the temperature to reach 34°C±.2°C.						
(w)	3.	Set Intoxilyzer mode selection in the ACA mode by switching mode selection switch #9 on or						
		selecting "C" on keyboard menu.						
(N)	4.	Attach simulator to the simulator entrance port on the Intoxilyzer.						
(w)	5.	Intoxilyzer 5000 display reads "READY TO START" or "PUSH BUTTON".						
(\mathcal{L})	6.	Push Start Test button or press enter on keyboard.						
(\mathbf{v})	7.	Insert card in response to display.						
$(\mathbf{r}_{\mathbf{r}_{\mathbf{r}_{\mathbf{r}}}})$	8.	Air blank completed.						
(UL)	9.	Calibration check completed. Test results 0. 99 AC						
(\mathbf{K}_{\bullet})	10.	Air blank completed.						
(1L)		When display reads Test Complete remove evidence card. Attach the card to the completed checklist.						
(IL)		Return mode selection switch #9 to off position after all calibration checks are complete or type Q and						
(M.,)		er on keyboard.						
SIGNA	rur.	E						
- WIIH	OK	WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD						

DHS/DSLS/Form C116 (Rev. 7-93)

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/13/2003 08:34
COPY NUMBER 1 OF 2

DFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TEST BRAC VALUE TIME
AIR BLANK .000 08:33
CAL. CHECK .099 08:34
AIR BLANK .000 08:34
OPERATOR:
ADDITIONAL INFORMATION:

EXHIBIT Q THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

AGENCY GOODY OAL D QA SPECIALIST WILL					
INTOXILYZER SERIAL # 66-004416 LOCATION DATE 4-13-03 19 TIME 9834					
Display reads "READY TO START" or "PUSH BUTTON TO START TEST."					
DIAGNOSTIC TESTS DVM Test check. Setting should be between .200 and .600. Mode selection switch S2 on, S1 and S3 off or keyboard menu selection "H". Reading is .37\(\) Display Test check. Mode selection switch S1 on and S2 and S3 off or keyboard menu selection "V". Printer Test check. Mode selection switch S1, S2, S3 off or keyboard menu selection "P". Clock time check. Mode selection switch S10 on or keyboard menu selection "E". Date check. Mode selection switch S11 on or keyboard menu selection "E".					
OPERATIONAL TESTS Alcohol-free subject Test result 0.000 AC Error recognition logic system functioning Invalid test printed Proper sample recognition system Invalid sample printed Deficient sample printed Completeness of sample purge with collection tube ** Calibration standard 0.100 AC Results: 0.099					
Instrument operating properly and accurately. YES NO					
SIGNATURE					
O. Alcohol concentration standard collection for subsequent analysis.**					
*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD					

**THIS STEP IS ONLY REQUIRED IF THIS DEVICE IS BEING USED FOR SAMPLE CAPTURE

DHS/DSLS/Form C117 (Rev. 7-93)

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/13/2003
COPY NUMBER 1 OF 1

08:40

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500

DVM TEST	- 79	
OPERATOR:	.310	
ADDITIONAL I	NFORMATION:	Managara Antonio and Antonio a

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/13/2003
COPY NUMBER 1 OF 1

08:41

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500

DISPLAY TEST

UPERAIUE: _		
ADDITIONAL	INFORMATION:	

SN 66-004416 DATE 04/13/2003 COPY NUMBER 1 OF 1 VERSION E 773.23 TIME 08:42

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500
ABCDEFGHIJKLMNOPQRSTUVWXYZ0123456789!@#\$
OPERATOR:
ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/13/2003
COPY NUMBER 1 OF 1

08:42

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500

TIME, DATE, LOC SET

OPERATOR:

ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT. INTOXILYZER - ALCOHOL ANALYZER AZ MODEL 5000 SN 66-004416 04/13/2003 08:43 COPY NUMBER 1 OF 2 SUB NAME=TEST, SQAP-AF, SUB DOB =04/13/30 SUB SEX =MALE WEIGHT =001 OFFICER =WILL, W, C OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500 TEST BRAC VALUE TIME AIR BLANK .000 08:43
SUBJECT TEST .000 08:43
AIR BLANK .000 08:43 OPERATOR: ADDITIONAL INFORMATION: ____

SN 66-004416 VERSION E 773.23 DATE 04/13/2003 TIME 08:44 COPY NUMBER 1 OF 2

INVALID TEST

SUB NAME=TEST, SQAP-BWT, SUB DOB =04/13/30 SUB SEX =MALE WEIGHT =001 OFFICER =WILL, W,C OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500

SAMPLE INTRODUCED AT IMPROPER TIME.
OPERATOR:
ADDITIONAL INFORMATION:

INVALID TEST

SUB NAME=TEST,SQAP-PB,
SUB DOB =04/13/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL,W,C
OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500
PRESSED START TEST AT
IMPROPER TIME
OPERATOR:
25. 99. 98. 98. 99. 99. 10. 10. 10. 10. 10. 10. 10. 10. 10. 10
ADDITIONAL INFORMATION:

SN 66-004416 DATE 04/13/2003 COPY NUMBER 1 OF 2

VERSION E 773.23 TIME 08:47

INVALID TEST

SUB NAME=TEST, SQAP-RFI, SUB DOB =04/13/30 UFFICER =WILL,W,U OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500

INHIBITED - RFI
OPERATOR:
ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/13/2003
COPY NUMBER 1 OF 2

08:48

SUB NAME=TEST, SQAP-MOA,
SUB DOB =04/14/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL, W, C
OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TEST BRAC VALUE TIME AIR BLANK .000 08:48

AIR	Bl	ANK		, ()()()	ļ		08:	48			
BEGI	N	NEW	DEPR	IVATI	ON I	PER	IOD	!			
OPER	άAT	OF: g	.,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,					***************************************	***************************************		
ADDI	TI	ONAL	INF	ORMAT	'ION	n H		***************************************			
***************************************					*************						
	*********		***************************************	***************************************		•••••••••••••••••••••••••••••••••••••••	***************************************	·	***************************************		
							***************************************			,	

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/13/2003
COPY NUMBER 1 OF 2

08:52

SUB NAME=TEST,SGAP-DS,
SUB DOB =04/14/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL,W,C
OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TEST BRAC VALUE TIME AIR BLANK .000 08:49 *SUBJECT TEST .087 08:52 AIR BLANK .000 08:52

* DEFICIENT SAMPLE - VALUE

PRINTED WAS HIGHEST OBTAINED.

THE MANUFACTURE OF THE PARTY OF THE PART
OPERATUR:
ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/13/2003 08:54
COPY NUMBER 1 OF 2

OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TEST BRAC VALUE TIME
AIR BLANK 000 08:53
CAL. CHECK 099 08:53
AIR BLANK 000 08:54

OPERATOR:

ADDITIONAL INFORMATION:

EXHIBIT Q THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

	\ INTOXILYZER MODE	L 5000*		
AGEN	CY (900 NY BYR P) QA SPEC	CIALIST _	WILL	
INTOX	CILYZER SERIAL # 66-004916 LOCATION	1	(Print Name)	
DATE	4-19 Pecos TIME /800			
	,			
1	Display reads "READY TO START" or "PUSH BUTTON	TO STAR	T TEST."	
DIAGN	JOSTIC TESTS			
1	DVM Test check. Setting should be between .200 and .60	0. Mode s	election switch S2 on, S1 and S	3 off
()	or keyboard menu selection "H". Reading is \(\frac{1}{376} \). Display Test check. Mode selection switch S1 on and S2 a	and S2 off	or keyboard many calcotion "\"	,
	Printer Test check. Mode selection switch \$1, \$2, \$3 off of			•
**	Clock time check. Mode selection switch \$10 on or keybo			
W	Date check. Mode selection switch S11 on or keyboard me			
OPERA	TIONAL TRETTS			
OPEKA	ATIONAL TESTS Alcohol-free subject Test result 0.000 AC			
$\frac{\omega}{\omega}$	Error recognition logic system functioning			
	Invalid test printed			
w	Proper sample recognition system			
	Invalid sample printed			
,	Deficient sample printed			
<u></u>	Completeness of sample purge with collection tube **			
Lactory To State of the State o	ent operating properly and accurately. YES $\sqrt{10}$ NO			
шышш	ent operating property and accurately. TES 100NONONO			
COMM	ENTS:			
			W. L	
	-4			
SIGNA	TURE			
0	Alachal assessmention at a dead callegation for subscript	*lai- 3	**	
<u>U.</u>	Alcohol concentration standard collection for subsequen	t allatysis.		
*WITH	OR WITHOUT VAPOR RECIRCULATION AND WI	TH OR W	ITHOUT KEYBOARD	
	S STEP IS ONLY REQUIRED IF THIS DEVICE IS BEI			

DHS/DSLS/Form C117 (Rev. 7-93)

INTOXILYZER - ALCOHOL ANALYZER AZ MODEL 5000 SN 66-004416 04/19/2003 17:58 COPY NUMBER 1 OF 2 OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500 BRAC VALUE TIME TEST .000 17:57 AIR BLANK ,000 ,000 .097 CAL, CHECK 17:58 AIR BLANK 17:58 OPERATOR: ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/19/2003
COPY NUMBER 1 OF 1

17:59

OFFICE	R BAD	(Ej <u>E</u> E == (0000	974
N.C.I.	C NO.	#AZ(0071	500

DVM	TEST		
OPER	ATOR:		
ADDI	TIONAL INFORMAT	rion:	

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/19/2003
COPY NUMBER 1 OF 1

17:59

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500 DISPLAY TEST

OPERATOR:

ADDITIONAL INFORMATION:

SN 66-004416 DATE 04/19/2003 COPY NUMBER 1 OF 1

VERSION E 773.23 TIME 18:00

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500
ABCDEFGHIJKLMNOPQRSTUVWXYZ0123456789!@#\$
OPERATOR:
ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/19/2003
COPY NUMBER 1 OF 1

18:00

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500

TIME,DATE,LOC SET

OPERATOR:	
ADDITIONAL INFORMATION:	٠.

INTUXILYZER - ALCUHUL ANALYZER
AZ MODEL 5000 SN 66-004416
04/19/2003 18:02
COPY NUMBER 1 OF 2

SUB NAME=TEST, SQAP-AF,
SUB DOB =04/19/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL,W.C
OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TEST BRAC VALUE TIME
AIR BLANK .000 18:01
SUBJECT TEST .000 18:01
AIR BLANK .000 18:02
OPERATOR:
ADDITIONAL INFORMATION:

SUB DOB =04/19/3	
	WEIGHT =001
OFFICER =WILL, W, (··•
OFFICER BADGE=000	
N.C.I.C NO.=AZOO	71500
PRESSED START TES	ST AT
IMPROPER TIME	
OPERATOR:	
ADDITIONAL INFORM	MATION:
·	

SN 66-004416 DATE 04/19/2003 COPY NUMBER 2 OF 2	VERSION E 7/3.23 TIME 18:03
INVALID TEST	
SUB NAME=TEST, SQAP-PB, SUB DOB =04/19/30 SUB SEX =MALE WEI OFFICER =WILL,W,C OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500	GHT =001
PRESSED START TEST AT IMPROPER TIME	
OPERATOR:	
ADDITIONAL INFORMATION	R II
nearranna sagaranna an	

SN 66-004416 DATE 04/19/2003 COFY NUMBER 1 OF 2

VERSION E 773.23 TIME 18:04

INVALID TEST

SUB NAME=TEST, SQAP-BWT,
SUB DOB =04/19/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL, W,
OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

AT IMPROPER TIME.
OPERATOR:
ADDITIONAL INFORMATION:

SN 66-004416 DATE 04/19/2003 COPY NUMBER 2 OF 2 VERSION E 773.23 TIME 18:04

INVALID TEST

SUB NAME=TEST,SQAP-BWT,
SUB DOB =04/19/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL,W,
OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

SAMPLE INTRODUCED AT IMPROPER TIME.

ADDITIONAL INFORMATION:	DEERATOR:			
	ADDITIONAL	. INFORMATION:		

SN 66-004416 VERSION E 773.23 DATE 04/19/2003 TIME 18:07 COPY NUMBER 1 OF 2

TIME 18:07

INVALID TEST

SUB NAME=TEST, SQAP-RFI, SUB DOB =04/19/30 SUB SEX =MALE WEIGHT =001 OFFICER =WILL, W, C OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500

INHIBITED - RFI

OPERATOR:

GOODYEAR POLICE DEPT. INTOXILYZER - ALCOHOL ANALYZER AZ MODEL 5000 SN 66-004416 04/19/2003 18:09 COPY NUMBER 1 OF 2 SUB NAME=TEST, SQAP-MOA, SUB DOB =04/19/30 SUB SEX =MALE WEIGHT =001 OFFICER =WILL, W, C OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500 TEST BRAC VALUE TIME AIR BLANK .000 18:08 INVALID SAMPLE .MOA 18:08 AIR BLANK .000 18:09 BEGIN NEW DEPRIVATION PERIOD OPERATOR: ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT. INTOXILYZER - ALCOHOL ANALYZER AZ MODEL 5000 SN 66-004416 04/19/2003 18:13 COPY NUMBER 1 OF 2 SUB NAME=TEST, SQAP-DS, SUB DOB =04/14/30 SUB SEX =MALE WEIGHT =001 OFFICER =WILL, W, C OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500 TEST BRAC VALUE TIME AIR BLANK .000 18:10
*SUBJECT TEST .096 18:12
AIR BLANK .000 18:13 * DEFICIENT SAMPLE - VALUE PRINTED WAS HIGHEST OBTAINED. OPERATOR: ADDITIONAL INFORMATION: INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
04/19/2003 18:17
COPY NUMBER 1 OF 2

OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TEST BRAC VALUE TIME
AIR BLANK .000 18:16
CAL. CHECK .000 18:17
AIR BLANK .000 18:17
OPERATOR:

ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.