# EXHIBIT P THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C R9-14-404(A)

#### ARIZONA DEPARTMENT OF HEALTH SERVICES

## STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 5000\*

#### STANDARD CALIBRATION CHECK PROCEDURE

Agency		2000 U EN 2 10 Date (-18-02 Time 1903
Intoxily	zer S	Serial # 66-004416 Location 600
QA Spe	cialis	st (V)((
		(Print Name)
.1.		
	Ι.	Pour a standard alcohol solution of known value into a clean dry simulator jar and assemble the
.1.		simulator. Insure that a tight seal has been made. Standard value: 0.100 AC
	2.	Turn on the simulator and allow the temperature to reach 34°C=.2°C.
(U)	3.	Set Intoxilyzer mode selection in the ACA mode by switching mode selection switch #9 on or
		selecting "C" on keyboard menu.
(11)	4.	Attach simulator to the simulator entrance port on the Intoxilyzer.
(M)	5.	Intoxilyzer 5000 display reads "READY TO START" or "PUSH BUTTON".
$(\mu)$	6.	Push Start Test button or press enter on keyboard.
(Ur)	7.	Insert card in response to display.
$(\mathcal{M})$	8.	Air blank completed.
$(\mathcal{W})$	9.	Calibration check completed. Test results 0.103 AC
$(\mathbf{W})$	10.	Air blank completed.
$(\mathcal{V})$	11.	When display reads Test Complete remove evidence card. Attach the card to the completed checklist.
(W)		Return mode selection switch #9 to off position after all calibration checks are complete or type Q and
		er on keyboard.
SIGNAT	TUR	E
*WITH	OR	WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD
DHS/DS	LS/I	Form C116 (Rev. 7-93)

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
05/18/2002 19:04
COPY NUMBER 1 OF 2

OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TEST BRAC VALUE TIME
AIR BLANK .000 19:08
CAL. CHECK .103 19:04
AIR BLANK .000 19:04

OPERATOR:
ADDITIONAL INFORMATION:

### 

#### ARIZONA DEPARTMENT OF HEALTH SERVICES

## STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 5000\*

AGENCY (060 DYEAR PD QA SPECIALIST WILL
INTOXILYZER SERIAL # (26 -004416 LOCATION GPO (Print Name)  DATE G-18-02 19 TIME
DATE (-18-62 P) TIME (90)
Display reads "READY TO START" or "PUSH BUTTON TO START TEST."
DIAGNOSTIC TESTS  DVM Test check. Setting should be between .200 and .600. Mode selection switch S2 on, S1 and S3 off or keyboard menu selection "H". Reading is 380.  Display Test check. Mode selection switch S1 on and S2 and S3 off or keyboard menu selection "V".  Printer Test check. Mode selection switch S1, S2, S3 off or keyboard menu selection "P".  Clock time check. Mode selection switch S10 on or keyboard menu selection "E".  Date check. Mode selection switch S11 on or keyboard menu selection "E".
OPERATIONAL TESTS  Alcohol-free subject Test result 0. Della AC  Error recognition logic system functioning  Invalid test printed  Proper sample recognition system  Invalid sample printed  Deficient sample printed  Completeness of sample purge with collection tube **  Calibration standard 0. Low ACResults: 0. NO  Instrument operating properly and accurately. YES NO
COMMENTS:
SIGNATURE
O. Alcohol concentration standard collection for subsequent analysis.**
*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD **THIS STEP IS ONLY REQUIRED IF THIS DEVICE IS BEING USED FOR SAMPLE CAPTURE

DHS/DSLS/Form C117 (Rev. 7-93)

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
05/18/2002 19:05
COPY NUMBER 1 OF 1

OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

DVM TEST

OPERATOR:
ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
03/18/2002
COPY NUMBER : OF 1

19:06

nr'	F	ï	CER	BADGE=0000974
Ν,	C	-:	I.C	NO.=AZ0071500

DISPLAY TEST

OPERATOR:
ADDITIONAL INFORMATION:

SN 66-004416 VERSION E 773.23 DATE 05/18/2002 TIME 19:06 COPY NUMBER 1 OF 1

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500
ABCDEFGHIJKLMNOPQRSTUVWXYZ0123456789!@#\$
OPERATOR:
ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
05/18/2002 19:08
COPY NUMBER 1 OF 2

SUB NAME=TEST,SGAP-AF,
SUB DOB =05/18/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL,W,C
OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TEST BRAC VALUE TIME
AIR BLANK .000 19:07
SUBJECT TEST .000 19:07
ATR BLANK .000 19:07
ATR BLANK .000 19:08

OPERATOR:
ADDITIONAL INFORMATION:

SN 66-004416 DATE 05/18/2002 COPY NUMBER 1 OF 2

VERSION E 773.23 TIME 19:09

INVALID TEST

SUB NAME=TEST,SQAP-PB,
SUB DOB =05/18/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL,W,C

OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500

PRESSED START TEST AT IMPROPER TIME

LJ1		. fr: 1	<u>i</u>	į	U.J	E. H		•••••									 		 	•••••	 	 *******	 
A)	ЭĽ	I	-	I	Oľ	VΑ	l	.]	N	FO	R	MA	T	ΙC	M	Rt II	 	********	 *******	***********	 	 *******	 ***
						•••••				**********	******	y11-8189				********	 ••••		 		 	 *******	 
			• • • • • •	••••		,	•••••				••••••						 		 ••••••	•••••	 •••••	 	 

SN 66-004416 VERSION E 773.23 DATE 05/18/2002 TIME 19:10 COPY NUMBER 1 OF 2

INVALID TEST

SUB NAME=TEST, SQAP-BWT, SUB DOB =05/18/30 SUB SEX =MALE WEIGHT =001 OFFICER =WILL, W, C OFFICER SADGE=0000974 N.C.I.C NO.=AZ0071500

SAMPLE INTRODUCED AT IMPROPER TIME.

OPERAT	문 a
ADDITI	NAL INFORMATION:

SN 66-004416 DATE 05/18/2002 COPY NUMBER 1 OF 2

VERSION E 773.23 TIME 19:11

INVALID TEST

SUB NAME=TEST.SGAP-RF1,
SUB DOB =05/18/30
SUB SEX =MALE WEIGHT =001
OFFICER =WILL.W.C
OFFICER SADGE=C000974
N.C.I.C NO.=AZ0071500

INHIBITED - RFI
OPERATOR:
ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT. INTOXILYZER - ALCOHOL AMALYZER AZ MODEL 5000 SN 66-004416 19:12 05/18/2002 COPY NUMBER 1 OF 2 SUB NAME=TEST, SQAP-MOA. SUB DOB =05/18/30 SUB SEX =MALE WEIGHT =001 OFFICER =WILL, W, C OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500 BRAC VALUE TIME TEST BRAC VALUE TIME AIR BLANK .000 19:12 INVALID SAMPLE .MOA 19:12 AIR BLANK .000 19:12 BEGIN NEW DEPRIVATION PERIOD OPERATOR: ADDITIONAL INFORMATION: GOODYEAR POLICE DEPT. INTOXILYZER - ALCOHOL ANALYZER AZ MODEL 5000 SN 66-004416 05/18/2002 19:17 COPY NUMBER 1 OF 2 SUB NAME=TEST, SQAP-DS, SUB DOB =05/18/30 SUB SEX =MALE WEIGHT =001 OFFICER =WILL, W, C OFFICER BADGE=0000974 N.C.I.C NO.=AZ0071500 BRAC VALUE TIME TEST AIR BLANK .000 19:13 \*SUBJECT TEST .092 19:16 AIR BLANK .000 19:16 \* DEFICIENT SAMPLE - VALUE PRINTED WAS HIGHEST OBTAINED. OPERATOR: ADDITIONAL INFORMATION: GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004416
05/18/2002 19:18
COPY NUMBER 1 OF 2

OFFICER BADGE=0000974
N.C.J.C NO.=AZ0071500

TEST BRAC VALUE TIME
AIR BLANK .000 19:17
CAL. CHECK .095 19:18
AIR BLANK .000 19:18

OPERATOR:

ADDITIONAL INFORMATION:

GOODYEAR POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 65-004218
05/18/2002 19:06
COPY NUMBER 1 OF 1

OFFICER BADGE=0000974
N.C.I.C NO.=AZ0071500

TIME, DATE, LOC SET

OPERATOR:
ADDITIONAL INFORMATION: